


PRC-Salttillo response to COVID-19 health concerns: [prc-salttillo.com/covid-19](https://www.prc-salttillo.com/covid-19)
(<https://www.prc-salttillo.com/covid-19>)

Community

 Topics

 Recent

 Archives

 (<https://salttillo.com/>) > Community (<https://salttillo.com/blog>) > News
(<https://salttillo.com/blog/topic/news>) > Tips for Cleaning & Disinfecting
Your PRC-Salttillo Device

Search Blog 



Tips for Cleaning & Disinfecting Your PRC-Salttillo Device



(<https://salttillo.com/blog/post/tips-for-cleaning-disinfecting-your-prc-salttillo-device>)



Just like hand washing, cleaning and disinfecting your PRC-Salttillo device on a regular basis is an important habit to get into. To aid in the fight against the spread of germs, here are some helpful steps for properly sanitizing your device and other equipment:

First Step: Put on protective gloves

Next Step: Clean the device and accessories

1. Wipe down the device and accessories with disinfectant wipes (Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, Fungicidal) like Metrex CaviWipes™. Follow the disinfectant product manufacturer's instructions for *cleaning*.
2. Take a Q-Tip with cleaner applied (for example, Windex® or another commercial cleaner) and clean inside all cracks and crevices. You may want to use a toothbrush or similar brush.
3. Blow the device off with an air hose or wipe it dry.

IMPORTANT! Make sure all soil is removed from the device and accessories. This is vital before proceeding to disinfecting the equipment.

Next Step: Disinfect the device and accessories

1. Wipe down the device and accessories with a new disinfectant wipe (Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, Fungicidal)—Metrex CaviWipes. Follow the disinfectant product manufacturer's instructions for *disinfecting*.
2. Make sure to clean inside all cracks and corners, and wipe more than once

if necessary to keep the device wet for a minimum of three (3) minutes.

3. Allow the device and accessories to dry.

Final Step: Wipe the touchscreen

1. When the device and accessories are dry, wipe down the viewing area (device touchscreen, TouchGuide, keyguard, etc.) with glass cleaner, so the screen doesn't discolor.
2. Allow the equipment to dry.

For questions about cleaning and disinfecting your device, we welcome you to contact your local PRC-Salttillo consultant. Find your representative ➔ **here**. (https://exploreaac.com/#whats_next)

Topics

News (<https://salttillo.com/blog/topic/news>)

AAC in Action

(<https://salttillo.com/blog/topic/aac-in-action>)

Resources & Tips

(<https://salttillo.com/blog/topic/resources-tips>)

Recent Posts

Tips for Cleaning & Disinfecting Your PRC-Salttillo Device

(<https://salttillo.com/blog/post/tips-for-cleaning-disinfecting-your-prc-salttillo-device>)

Overcoming AAC Resistance: Communication is Key!

(<https://salttillo.com/blog/post/overcoming-aac-resistance-communication-is-key>)

Playground Core Communication Boards from PRC-Salttillo

(<https://salttillo.com/blog/post/playground-core->

Archives

2020

March

(<https://salttillo.com/bl>)

February

(<https://salttillo.com/bl>)

January

(<https://salttillo.com/bl>)

2019

2018

2017

2016

2015

2014

2013

2012

2011

communication-boards-from-prc-salttillo)

PRC-Salttillo Named "Business of the Year"
(<https://salttillo.com/blog/post/prc-salttillo-named-business-of-the-year>)

Your 2.23 Chat Software Update Awaits!
(<https://salttillo.com/blog/post/your-223-chat-software-update-awaits>)

Register Today for January 2020 Salttillo Webinars!
(<https://salttillo.com/blog/post/register-today-for-january-2020-salttillo-webinars>)

Connect with us!



2143 Township Road 112

Millersburg OH 44654

☎ 1-800-382-8622 (tel:+18003828622)

✉ info@salttillo.com

(mailto:info@salttillo.com)



(<https://salttillo.com/blog/post/overcoming-aac-resistance-communication-is-key>)

The Latest

Tips for Cleaning & Disinfecting Your PRC-Salttillo Device

(<https://salttillo.com/blog/post/tips-for-cleaning-disinfecting-your-prc-salttillo-device>)

Overcoming AAC Resistance:

Communication is Key!

(<https://salttillo.com/blog/post/overcoming-aac-resistance-communication-is-key>)

Playground Communication Boards from PRC-Salttillo

(<https://salttillo.com/blog/post/playground-communication-boards-from-prc-salttillo>)

core-communication-boards-from-prc-salttillo)

PRC-Salttillo Named "Business of the Year"
(<https://salttillo.com/blog/post/prc-salttillo-named-business-of-the-year>)

Your 2.23 Chat Software Update Awaits!
(<https://salttillo.com/blog/post/your-223-chat-software-update-awaits>)

Read More... (<https://salttillo.com/blog>)

Looking for Help?

Find your Representative
(<https://salttillo.com/ reps>)

Get Help with Funding
(<https://salttillo.com/ funding>)

Support Articles
(<https://salttillo.com/ support>)

Trouble Tickets (<https://salttillo.com/ tt>)

Information

TouchChat (<https://touchchatapp.com>)

Silver Kite (<http://www.silver-kite.com>)

Newsletter
(<https://salttillo.com/ newsletter>)

Careers (<https://salttillo.com/ careers>)

Terms & Conditions
(<https://salttillo.com/ terms-and-conditions>)

Privacy (<https://salttillo.com/ privacy-policy>)

Trademarks
(<https://salttillo.com/ trademarks>)